CLASSIC 2

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Product model : CL2
Version : V1.3

Bluetens France 
5 passage Saint Bernard 75011 Paris, France.

Bluetens limited 
Unit G, 15/F, TAL Building, 49 Austin Road, Kowloon, Hong Kong
Contact : cs@bluetens.com

S4M EUROPE SAS 
59, rue Castellion 01100 Oyonnax FRANCE - regulation@s4m-europe.com

English	4
Français	23
Deutsch	42
Español	61
Other languages	87
Electrode placements	84
Description of currents output	86
Electromagnetic compatibility	88

Welcome !

The Classic 2 electrostimulator is a medically certified device that sends electrical impulses to muscles and nerves to relieve pain, relax or strengthen muscles.

Read the instructions of use carefully to make sure you use the device correctly.

BENEFITS OF ELECTROSTIMULATION:

TENS

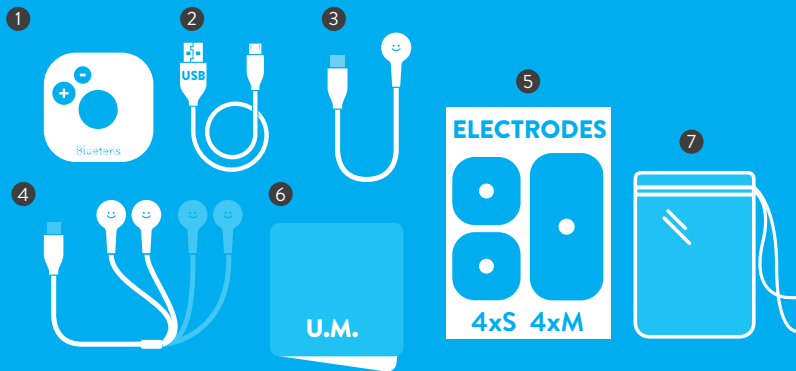
The device is made to relieve acute or chronic pain. Totally natural, it is an effective pain management solution. The Classic 2 uses 2 modes of stimulation to relieve pain, one with an immediate effect, called «Gate Control», and a slower one, longer-lasting called «Burst».

This method has been used for many years in pain centers, hospitals and physiotherapy practices.

EMS

Electrostimulation is used in the medical and sports sectors, to act on muscles in order to relax or strengthen different body areas. Physical preparation, targeted muscle reinforcement, treatment and muscle recovery form a complete body workout to ensure good muscular health.

This user manual is available by following the link:
<https://www.bluetens.com/en/content/48-user-manuals>

- 1 Bluetens Classic 2 (blue) CL2 (x1)
- 2 USB A to USB C cable white 80cm (x1, PN: V2C-A1C080WH)
- 3 Electrode cable 1 output 25cm (x1, PN: V2E-C1S025WH)
- 4 Electrode cable 4 output 110cm (x1, PN: V2E-C4S110WHBU)
- 5 Electrodes pack Mix: 4S (50x50mm) + 4M (100x50mm), PN: MG-MIX2
- 6 User manual (x1)
- 7 Protective bag

EN PRECAUTIONS FOR USE

Target patient population: (Adults: 18 years old and above)

- EMS Function Indications: Healthy individuals seeking to enhance muscle strength and improve athletic performance; patients with restricted mobility due to post-surgical recovery or prolonged bed rest who require prevention of muscle atrophy (prior medical evaluation is necessary to confirm absence of contraindications).
- TENS Function Indications: Patients suffering from acute or chronic pain (e.g., neck, shoulder, back, or leg pain), pelvic pain, primary/secondary dysmenorrhea, or endometriosis-related pain (with confirmed pain etiology and medical clearance to ensure absence of contraindications).

Eligible Operator Population:

- Must be 18 years old or above, with full civil capacity;
- If the patient operates the device themselves: Must have basic reading and comprehension skills to understand the instruction manual and follow steps, and be able to independently report physical sensations (e.g., pain, numbness) during use;
- If operated by medical staff/family members: Must be familiar with device functions and operating procedures, have basic safety knowledge of electrical stimulation, be able to assist patients in adjusting device parameters, and monitor the patient's physical state during use.

Contraindicated Operator Population

- Minors under 18 years old;
- Individuals with visual, auditory, or physical impairments that prevent accurate device operation;
- Individuals lacking basic safety awareness and unable to identify abnormal conditions during patient use

MAIN DEVICE:

- The electrode is treated as a BF-type applied part.
- Device name: Bluetens Classic 2
- Product model: CL2
- Weight: 33 grams
- Battery voltage: 3.7V 500mAh lithium-ion polymer battery
- USB charging: Type C 5V/500mA
- Frequency: 1Hz to 1200Hz
- Output pulse: 24 to 400uS
- Output voltage: 0V to 120V (at 500Ω load)
- Output power: 10mW (r.m.s) MAX
- Operating environment:
 - Temperature: 5°C to 40°C
 - Humidity: 30%RH to 80%RH
 - Atmospheric pressure: 700 to 1060hPa
- Storage environment:
 - Storage temperature: -20°C to 70°C
 - Humidity: 30%RH to 80%RH
 - Atmospheric pressure: 700 to 1060hPa
- Product dimensions: 50.8mm X 51.5mm X 18mm
- Minimum lifespan: 5 years
- Software version: V3.3

ELECTRODES:

- Support material: non-woven fabric
- Safety: Biocompatibility of the G607 gel used in this product is compliant with ISO 10993-5, ISO 10993-10 standards
- Lifespan: sealed storage for 3 years, to be stored at room or cool temperature (5°C to 35°C) and protected from direct sunlight.

CONTRAINDICATIONS

1. Do not use this device if you have a pacemaker, implanted defibrillator, or any other implanted metallic or electronic device. Such use could cause electric shock, burns, electrical interference, or death.
2. The device should not be used when cancerous or other lesions are present in the treatment area.
3. Stimulation should not be applied over open wounds, rashes, or swollen, red, infected, or inflamed areas (e.g., phlebitis, thrombophlebitis, varicose veins, obliterative arteriosclerosis, etc.).
4. Electrode placements should be avoided in the carotid sinus region (front of the neck) or transcranially (across the head).
5. Anxious users - the use of electrical stimulation requires user cooperation; therefore, the procedure should not be attempted in users with communication disabilities or mental incapacity.
6. Users with cerebrovascular issues - individuals with a history of aneurysm, stroke, and transient ischemia should not undergo electrical stimulation, as it stimulates peripheral blood flow, which could be fatal in such cases.
7. Epileptic users - Electrical stimulation impulses have the potential to trigger a seizure.
8. Cases of acute pain/pain of unknown origin - using an electrical stimulator in undiagnosed cases may hinder diagnosis.
9. Acute inflammation and swelling of the genitourinary system, tumors, urinary stones, history of nervous system diseases, and other illnesses.
10. Do not use during pregnancy, especially during the first trimester.
11. Presence of a severe systemic disease, such as scleroderma, diabetes mellitus with severe complications, myopathy, multiple sclerosis, Parkinson's disease, myelopathy, or serious diseases of the liver, lungs, kidneys, hematological system, or other comorbidities.

WARNINGS

The use of the device should be discontinued in the following circumstances:

1. If you feel unwell or uncomfortable, stop immediately. If there are any abnormalities in the skin or body, consult your doctor and follow their instructions.
2. If you wish to move the electrode to other parts of the body during operation, please turn off the power before moving it. Failure to do so may result in an electric shock.
3. Do not use this device while wearing other electronic devices.
This could cause malfunction of these devices (e.g., malfunction of an electronic watch).

4. As subcutaneous use presents a high risk to humans or other devices, the use of this device is prohibited.
5. Use under supervision mandatory for children (over 7 years old) and disabled persons. Risk of accident or discomfort.
6. Do not use this appliance in a bathroom or a place with high humidity. Do not use while bathing or showering. Strong electric shock could occur.
7. Do not use while sleeping. This could damage the product, and the displacement of the electrode could cause discomfort.
8. Do not use while driving. It could cause an accident due to the strong stimulation.
9. Do not keep the electrodes on your body all the time. This could cause skin irritation.
10. Do not use metal to touch the electrode. This could cause an electric shock.
11. Application near the thoracic region could cause cardiac discomfort. Seek the advice of a doctor beforehand.
12. If in doubt, consult your doctor.
13. Discontinue use and do not increase the intensity level if you experience discomfort during use.
14. The electrode cable must not encircle your neck to avoid strangulation.
15. Do not modify the equipment. Please use original accessories, it will cause an unusable effect if you use third party accessories on the market.
16. Keep out of reach of children to avoid inhalation or ingestion of small parts.

PRECAUTIONS FOR USE

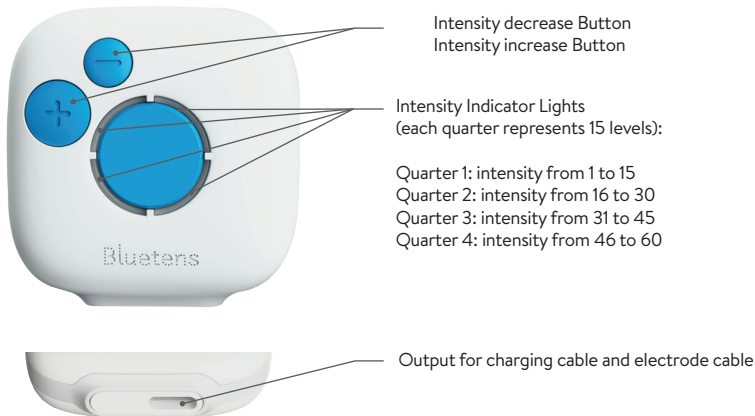
1. If you have a skin allergy, you should stop using it immediately.
2. Stimulation must not be applied through or over the head, directly to the eyes, covering the mouth or on the front of the neck (especially the carotid sinus).

SIDE EFFECTS

1. Users may experience skin irritation and a burning sensation under the stimulation electrodes applied to the skin.
 2. Users may experience pain or discomfort during or after application.
- Note: Users should stop using the device and consult their doctor if they experience any adverse reactions.

THE DEVICE

The Bluetens Classic 2 turns on and off by pressing the blue button in the center of the device. Here is an explanation of the inputs, outputs, and buttons of the device.

CHARGING THE BATTERY

The Classic 2 is charged using the USB-C cable included in the box.

Charging time is approximately 1 hour.

When the battery is below 20%, the charge indicator light flashes orange.

This indicator light will remain orange during the charging time and will go out once the battery is full.

We recommend charging the device with its USB-C cable connected to a computer.

If you wish to use a mains adapter, please ensure that the power is compatible.

The battery recharges on average once a month, if the device is used once every 2 days.

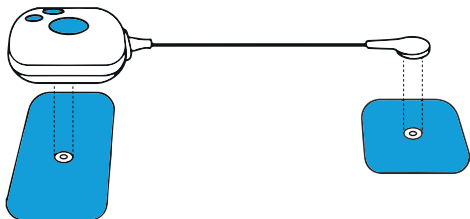
Note:

- This device cannot be used while charging.
- The mains adapter must comply with standard IEC62368-1, respect at least 2 MOOP and have an output of 5V/1A.


CONNECTING THE ELECTRODES

Here's how to connect the electrodes:

1. Short electrode cable + 2 electrodes (S or M)

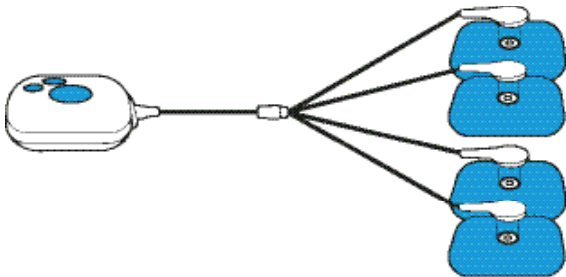2. Long electrode cable + 4 electrodes (S or M)

DOWNLOAD AND INSTALLATION

The Bluetens Application is available on Google Play and Appstore.

The Application is compatible with Smartphones and Tablets:

iPhone : iOS 13 and above

Android : Android 10 and above

(Modified Android versions may make the Application incompatible)

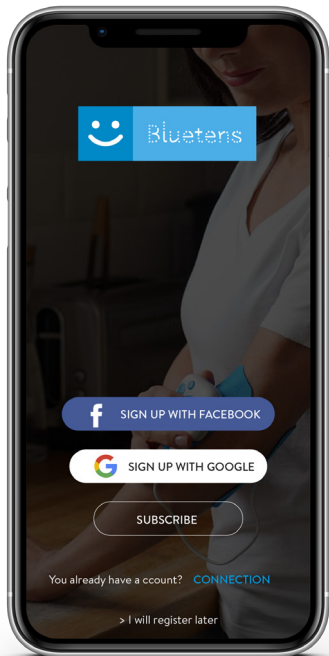
Download and install the Application according to your Smartphone.

Notes:

Bluetooth must be activated to use the Application.


Android smartphones must also authorise access to the location service, to the Application.

The Bluetens Application does not have access to any data. This condition is required by Android versions to stabilise Bluetooth.

FIRST USE

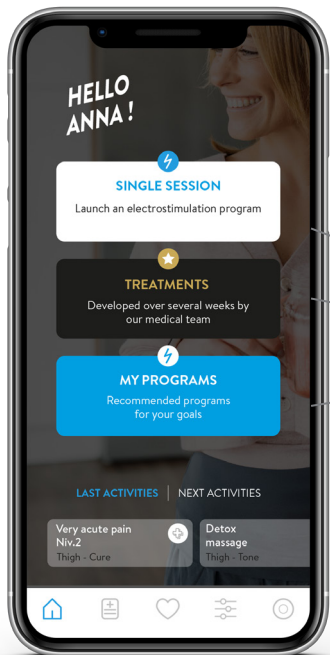- > Launch the Application and select your device from the available options.
- > Log in using the options provided or create an account.
- > A tutorial will introduce you to the Application and its features.


PRESENTATION AND USE

> Choose from the main functions on the home page:

SIMPLE ELECTROSTIMULATION SESSION

TREATMENTS: generic treatments for common pathologies

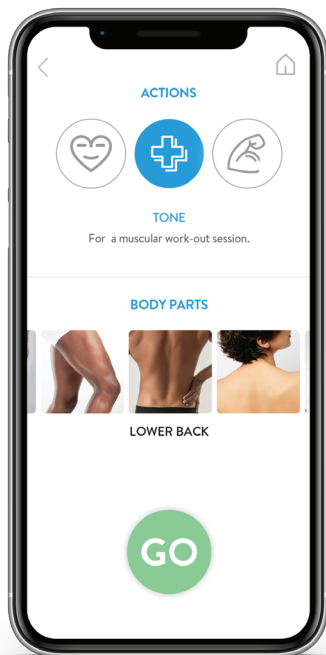MY PROGRAMS: recommended programmes to achieve the goals you have set.

SINGLE SESSION: START A PROGRAMME

> **Select the action and the body part**

CURE

Different programmes to relieve all types of pain.

RELAX

Muscle massage programmes and post-physical exertion recovery.

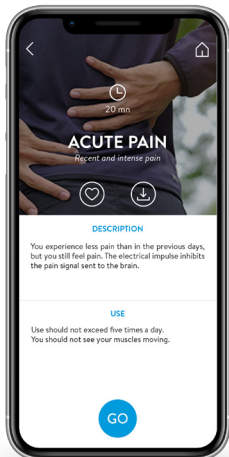
TONE

Physical preparation and muscle strengthening programmes.

Choose the body parts according to your needs.

> **Press «GO» to access the list of programmes**


PROGRAMME DESCRIPTION


ELECTRODES PLACEMENT

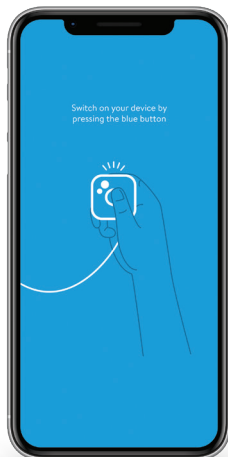TURN YOUR DEVICE ON

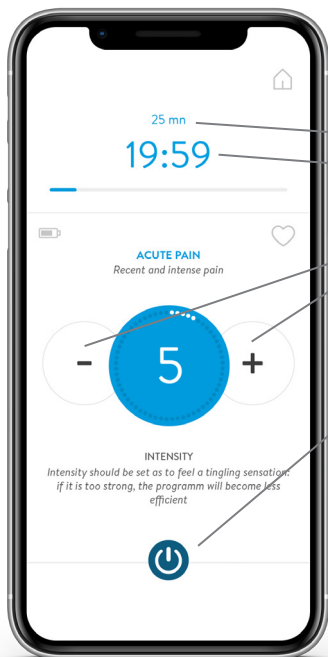**The connection between the Application and the device
is established automatically at this stage.
Do not pair manually!**

RUNNING PROGRAM

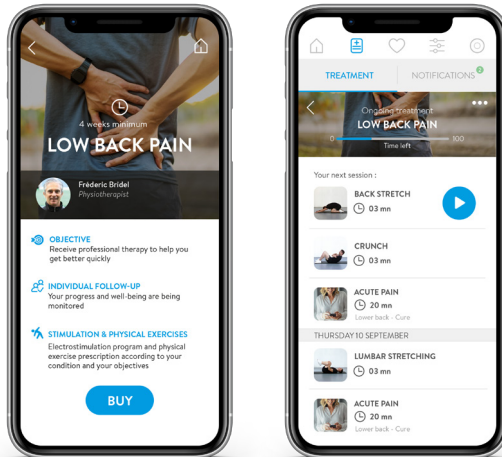
TOTAL PROGRAMME DURATION
REMAINING TIME COUNTDOWN

The default intensity is 1, press + or - to change it. The maximum intensity is 60. The intensity can also be set directly on the device.

PRESS THIS BUTTON TO STOP THE PROGRAM

Note:

Once your session is over, make sure you switch off the device before handling the electrodes. Replace them on the plastic (indicated side) and store them in a cool, dry place.

PREMIUM FUNCTION PERSONALISED SUPPORT TO HELP YOU REACH
YOUR SPORT OR HEALTH OBJECTIVES

- > Select or describe your problem, then fill in your profile and proceed to payment. Within 48 hours you will receive your personalised treatment to follow for a minimum of 4 weeks (electrostimulation and physical exercises) and follow the prescriptions to get better quickly. The healthcare professional will make sure that your exercises are carried out properly and that you feel good.

MAIN DEVICE

- Remove the cables from the device after use.
- Use a soft cloth to clean dirt on the device
- Use a damp cloth with a neutral detergent or alcohol to clean if necessary.
- Do not use thinner or petrol to clean.
- Do not use alcohol on the back of the device
- Do not use any other detergents.
- Keep water or liquids away from the device
- Keep out of direct sunlight, heat, damp and dusty places.

ELECTRODES

- Do not use extra gel with the electrodes.
- Always replace and store the electrodes on the correct side of the plastic provided.
- Use a little water to clean the electrode if necessary.
- Do not wash.
- Do not wipe the electrode with a cloth or paper.
- The lifespan of the electrodes varies between 30 and 50 uses depending on the quality of the skin, hairiness, perspiration and the care taken.
- Electrodes are consumables; when they no longer adhere to the skin, it's a sign that they need changing.
- Do not stack or bend the electrodes.
- Do not touch the gel with your fingers.
- Apply to clean, dry skin, without cream or oil.

Essential informations for a correct use is indicated by the corresponding symbols. The following symbols can be seen on the device and on the labelling.

SYMBOL	TITLE
IP22	IP code of the device: the degree of protection of this device against the intrusion of solid foreign objects -- ≤ 12.5 mm in diameter (and against access to dangerous parts with a finger); the degree of impermeability is of the drip type (inclination of 150).
	Disposal in compliance with directive 2002/96/EC (WEEE)
	This side up
	Date of manufacture
	Batch code
	CE mark, Certificate issued by TUV SUD.
	Manufacturer
	Type BF applied part
	Warning: to avoid the risk of injury, read the manufacturer's instructions carefully.
	Symbol for «AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY»
	Fragile, handle with care
	Store the product in a dry place. Protect from water and rain.
	Product packaging is recyclable
	Importer
	Medical device

The product must not be disposed of with household waste.
Consumers are legally obliged to dispose of electrical equipment at a collection point in their town.
Contact your local council for more information or a retailer.

DO YOU NEED HELP?

Our customer service team deals directly with all requests for advice or after-sales service. Our agents are fast and available by e-mail at cs@bluetens.com.

We'll get back to you within a day (excluding weekends and public holidays).

Online troubleshooting: <https://www.bluetens.com/en/content/28-support-bluetens>

The CL2 is guaranteed for two years. This guarantee does not cover damage caused by improper maintenance or use that does not comply with the relevant documentation, accidents (impact, fire, liquid), modifications, repairs, misuse or normal wear and tear. This warranty covers defects in materials or workmanship. This warranty only covers products purchased directly from Bluetens or an authorized Bluetens reseller.

In the event of a defect covered by the warranty, Bluetens will send you a return slip and will be responsible for the repair or exchange of your device as well as its return.

WARRANTY PERIOD

Your device is guaranteed for 2 years from the date of receipt of the product if purchased online, or from the date of purchase if purchased from a retailer.

NOT COVERED BY THE WARRANTY

Electrodes, cables and accessories included in the box are not covered by this warranty.

WARRANTY DISCLAIMER

Bluetens has not authorised any agency to repair its products. If you experience any problems with your device, please contact us. Bluetens will not be responsible for the results of any repair. The intervention of a third party on our products will void the 2-year warranty. The use of third-party electrodes or accessories could cause the device to malfunction and will void the warranty. The user should not attempt to repair the device or its accessories by himself. Each device has been inspected through systematic validation. The performance is stable and does not require calibration or validation. If your product does not achieve the expected performance and the basic function changes during normal use, please contact us. Please note that Bluetens will not be responsible for quality issues caused by improper use.

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RECOMMENDED ELECTRODE PLACEMENTS

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Here is an overview of the possible electrode placements for the different cables and electrodes, on various parts of the body.

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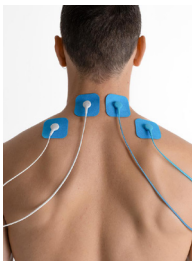
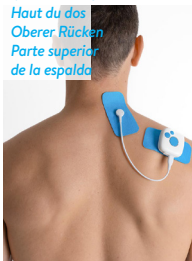
DESCRIPTION OF CURRENTS OUTPUT

Instant electric voltage/current when using Electrodes:

Load 500Ω: MAX 120V / MAX 240mA

Load 1000Ω: MAX 150V / MAX 150mA

TENS

N°	PROGRAM	OUTPUT PARAMETERS
1	Acute Pain	Frequency: 80Hz - Impulse: 150µs - Treatment time: 20 minutes
2	Very acute pain	Frequency: 100Hz - Impulse: 250µs - Treatment time: 20 minutes
3	Cervical Pain	Frequency: 2Hz - Impulse: 200µs - Treatment time: 20 minutes
4	Intense recovery 1	Frequency: 2-4-6-8-10-9-7-5-3-1 Hz - Each step 3 minutes Impulse: 150µs - Treatment time: 30 minutes
5	Endorphin 1	Frequency: 5Hz - Impulse: 250µs - Treatment time: 30 minutes
6	Chronic pain 1	Frequency: 80Hz - Impulse: 200µs - Treatment time: 60~120 minutes
7	Decontracting massage	Frequency: 1Hz - Impulse: 150µs - Treatment time: 30minutes
8	Active recovery 1	Frequency: 10-9-8-7-6-5-4-3-2-1 Hz - Each step 3 minutes Impulse: 150µs - Treatment time: 30 minutes

N°	PROGRAM	OUTPUT PARAMETERS
1	Strengthening n°1	Warm-up: Frequency: 8Hz - Impulse: 150µs 5 minutes Training: Frequency variable - Impulse: 150µs 8Hz~50Hz Inc 1.5 seconds - 60Hz Hold 6 seconds 60Hz~8Hz Dec 1.5 second - 8Hz Hold 6 Seconds Phase time est: 15 second - Loop for: 25 minutes Cool-down: Frequency: 3Hz Impulse: 300µs 5 minutes Treatment time: 35 minutes
2	Strength 2s	Sequence: Frequency: 50 Hz - Impulse variable Impulse: 50~250µs inc 1.5 seconds Impulse: 250µs hold 2 seconds Impulse: 200~65µs dec 1.5 seconds Pause: 4 seconds - Loop for: 20 minutes
3	Muscle sharpening 1	Warm-up: Frequency: 6Hz - Impulse: 150µs 5 minutes Training: Frequency variable - Impulse: 150µs 6Hz~30Hz Inc 4 seconds - 30Hz Hold 10 seconds 30Hz~4Hz Dec 1 second - 4Hz Hold 6 Seconds Phase time est: 21 second - Loop for: 25 minutes Cool-down: Frequency: 3Hz - Impulse: 300µs 5 minutes Treatment time: 40 minutes
4	Muscle tone 1	Warm-up: Frequency: 100Hz - Impulse: 150µs 5 minutes Training: Frequency: 4Hz~20Hz~4Hz Inc/Dec step by 1Hz Impuse 150 Phase time est: 22.5 second - Loop for: 20 minutes Cool-down: Frequency: 3Hz - Impulse: 150µs 5 minutes Treatment time: 30 minutes
5	Prevention muscle loss 1	Frequency variable - Impulse: 200µs 15Hz~80Hz Inc 2.2 seconds - 80Hz Hold 10 seconds 80Hz~15Hz Dec 1.3 second - 15Hz Hold 8 Seconds Phase time est: 21.5 second - Loop for: 30 minutes

ELECTROMAGNETIC COMPATIBILITY

The device has been thoroughly tested and inspected to assure proper performance and operation.
The EMC Test Report Number of TUV: 68.730.20.0074.01
Model: CL2, CL2M, CL2P, DS2

TEST CONFIGURATION

Configuration	Description
AC Powered	230VAC/50Hz
AC Powered	100VAC/50Hz & 240VAC/50Hz (Only for Voltage dips and Voltage interruptions)

OUTPUT PARAMETERS

Output Frequency	1Hz	10Hz	50Hz	100Hz	200Hz	500Hz	1KHz	1,2KHz
Output Pulse	400uS	300uS	100uS	100uS	50uS	50uS	34uS	24uS
Output Voltage Vpp	0-120V	0-120V	0-110V	0-85V	0-85V	0-57V	0-57V	0-57V

MODES OF OPERATION:

Mode	Description
Charging	The EUT's USB-C port is connected to a single-port USB-C cable (length: 80 cm) and is charged by an AC / DC adapter.
EMS + Lien BT	The EUT is powered by a rechargeable 3.7VDC lithium battery. Its USB-C port is connected to a double electrode cable (4 ends, length: 110 cm), and the electrode cable terminals are connected to electrode pads (Size: 50 x 100 mm). The part of the electrode cable 40 cm away from the EUT is immersed in 0.9% saline solution. The EUT is connected to the APP via Bluetooth, and its treatment functions are controlled by the APP or by the EUT itself. Next, the EUT is set to Nev.1 Acute Pain mode (Pain) and set to maximum power for the operation.
TENS + Lien BT	The EUT is powered by a rechargeable 3.7VDC lithium battery. Its USB-C port is connected to a dual-electrode cable (4 ends, length: 110 cm), and the electrode cable terminals are connected to electrode pads (Size: 50 x 100 mm). The part of the electrode cable that is 40 cm away from the EUT is placed in 0.9% saline solution. The EUT is connected to the APP via Bluetooth, and its processing functions are controlled by the APP or by itself. Next, the EUT is set to TENS mode and adjusted to maximum power for operation.

EMISSION TESTS EN 60601-1-2:2015, IEC 60601-1-2:2014

Test Condition: Group1, Class B	Test Result
Radiated Emission 30MHz to 1000MHz	Pass
Conducted Emission on AC 150kHz to 30MHz	Pass
Harmonic Class A	Pass
Flicker	Pass

Test Condition: Group1, Class B	Test Result
Electrostatic Discharge (IEC 61000-4-2) ±2kV ±4kV ±6kV ±8kV ±15kV	Contact: ±8kV Air: ±2kV ±4kV ±8kV ±15kV
Radiated Immunity (IEC 61000-4-3) 80MHz to 2700MHz 10V/m 385MHz 27V/m 450MHz 28V/m 710MHz, 745MHz, 780MHz 9V/m 810MHz, 870MHz, 930MHz 28V/m 1720MHz, 1845MHz, 1970MHz 28V/m 2450MHz 28V/m 5240MHz, 5500MHz, 5785MHz 9V/m	10V/m, 80% Am at 1kHz 27V/m PM at 18Hz 28V/m FM ± 5 kHz deviation at 1kHz sine 9V/m PM at 217 Hz 28V/m PM at 18Hz 28V/m PM at 217 Hz 28V/m PM at 217 Hz 9V/m PM at 217 Hz
Electrical Fast Transient (IEC 61000-4-4) For AC Power port: ±2kV, 100 kHz repetition frequency For signal input/output port: ±1kV, 100 kHz repetition frequency	For AC port: Power supply lines: ±2kV
Surge (IEC 61000-4-5) For AC Power port: ±0.5kV, ±1kV (Line to line) ±0.5kV, ±1kV, ±2kV (Line to Ground) For signal input/output port: ±2kV (Line to Ground)	For AC port: (L-N): ±0.5kV ±1kV
Conducted Immunity (IEC 61000-4-6) For AC Power port, patient coupling port and signal input/output port: 150KHz to 80MHz 3V (80% Am at 1kHz) ISM and amateur radio bands between 150KHz to 80MHz 6V (80% Am at 1kHz)	For AC port and patient coupling port: 150KHz to 80MHz 3V (80% Am at 1kHz) ISM and amateur radio bands between 150KHz to 80MHz 6V (80% Am at 1kHz)
Voltage Dips and Interruption (IEC 61000-4-11) 0%, 70%, 0% of UT	For AC power port: 0% for 0.5 cycle (at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°), 0% for 1 cycle (at 0°), 70% for 25 cycles (at 0°), 0% for 250 cycles
Power Frequency Magnetic Field (IEC 61000-4-8) 50Hz, 60Hz 30A/m	50Hz: 30A/m 60Hz: 30A/m

OTHER LANGUAGES

**THIS USER MANUAL IS AVAILABLE IN OTHER LANGUAGES
AT THE FOLLOWING LINK:**

<https://www.bluetens.com/en/content/48-user-manuals>

Take care of yourself

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Cuidese

www.bluetens.com